

DETACHOL® BOTTLE


APPLICATION INSTRUCTIONS



Safety, comfort and convenience come in four easy steps:



Saturate

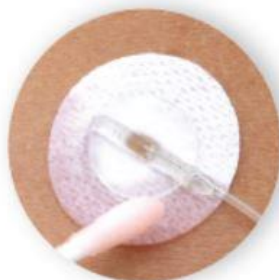

Apply



Remove



Cleanse

Saturate cotton swab or gauze with Detachol and apply to the edge of tape or device. Once the corner is permeated, gently lift it up. Continue to apply Detachol while gently pulling the device until the adhesive is completely removed. Wash the area with soap and water or skin cleanser. *See container for full instructions for use

CAUTIONS • For external use only • Avoid contact with broken skin • Flammable

safe for all skin types > alcohol free > acetone free > non-irritating

To purchase visit

shop.eloquesthealthcare.com

Eloquest
HEALTHCARE®